

Expressew | Flexible suture passer


Jaw Ratchet/Ratchet Release

mechanism provides "single finger" operation, control and simplicity


Needle Auto Retraction feature provides simplified "single step" deployment and retraction


Simplified Needle and Suture loading saves time, increasing Operating room efficiency

Polymer coated needle increases reliability by reducing needle deployment and retraction forces

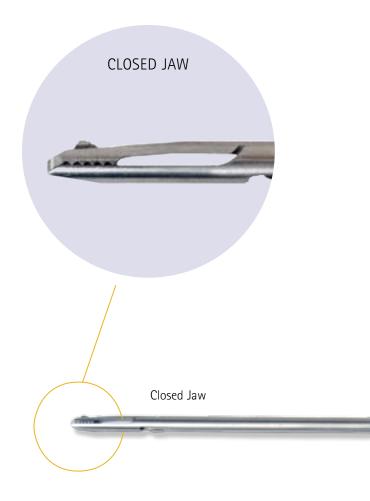


Intuitive. Simple. Flexible. Reliable.

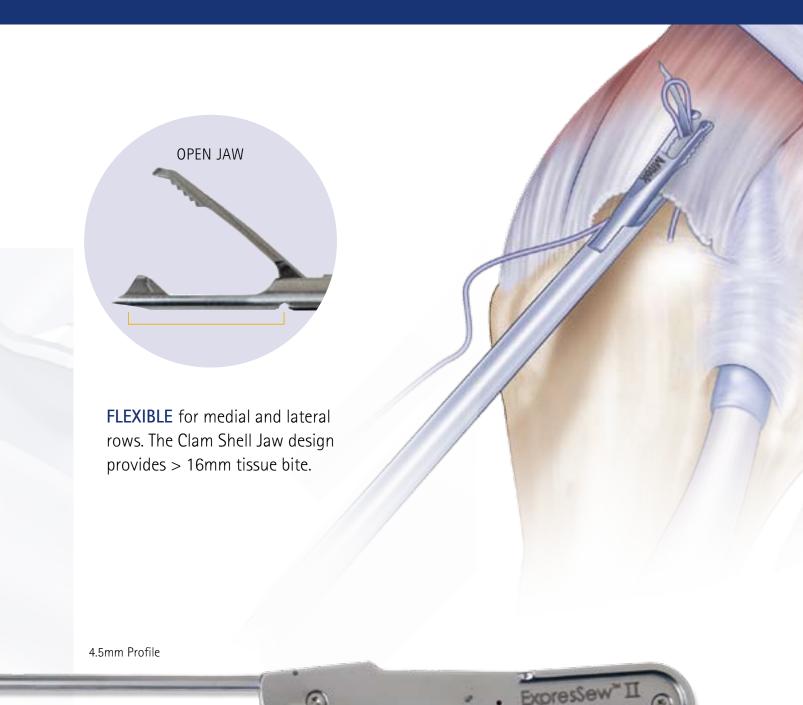

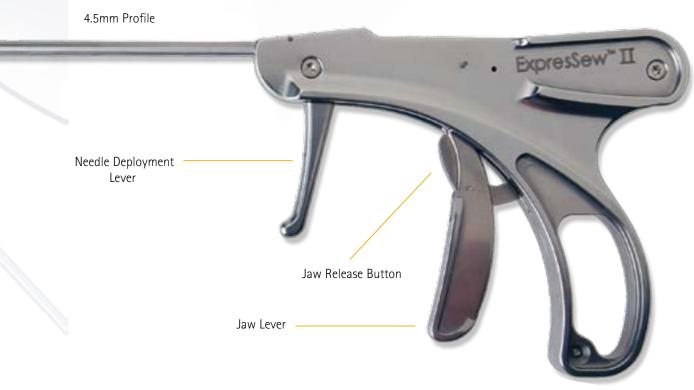


SURGICAL TECHNIQUE

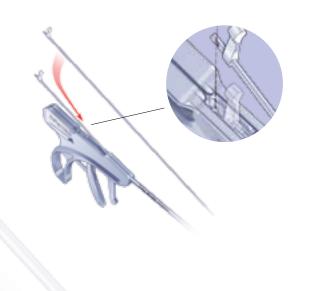

1. Load the needle by grasping the needle flag and sliding the needle through the back end of the device. Align the notch on the needle with the nub in the needle rack and drop the needle into place. Test fire to confirm the needle is properly loaded.




2. Load the suture, making a loop of suture that creates a short 1" tail. Load the short tail down into the front loading slot and slide it to the back end. Insure the suture is securely seated in the most proximal aspect of the suture slot.

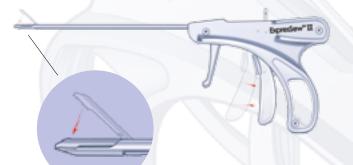

Close the jaw in preparation for use by pulling the jaw lever.

EXPRESSEW II FLEXIBLE SUTURE PASSER

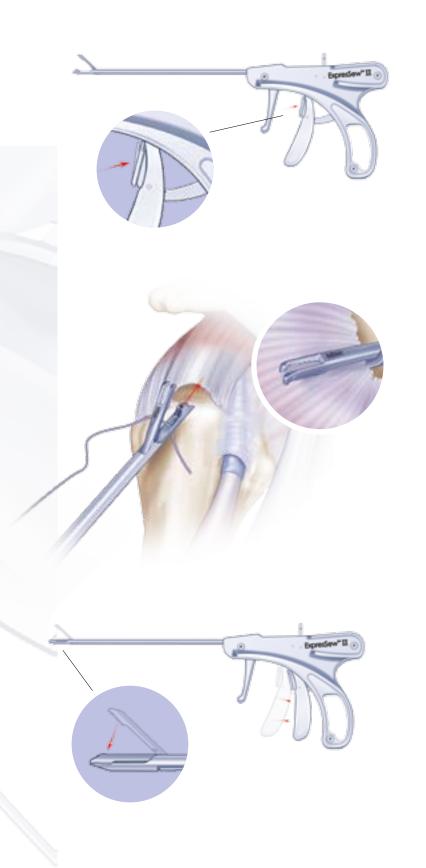


3. Open the jaw by pushing the jaw release button.

Engage the tissue.

Close the jaw by pulling the jaw lever.




SURGICAL TECHNIQUE

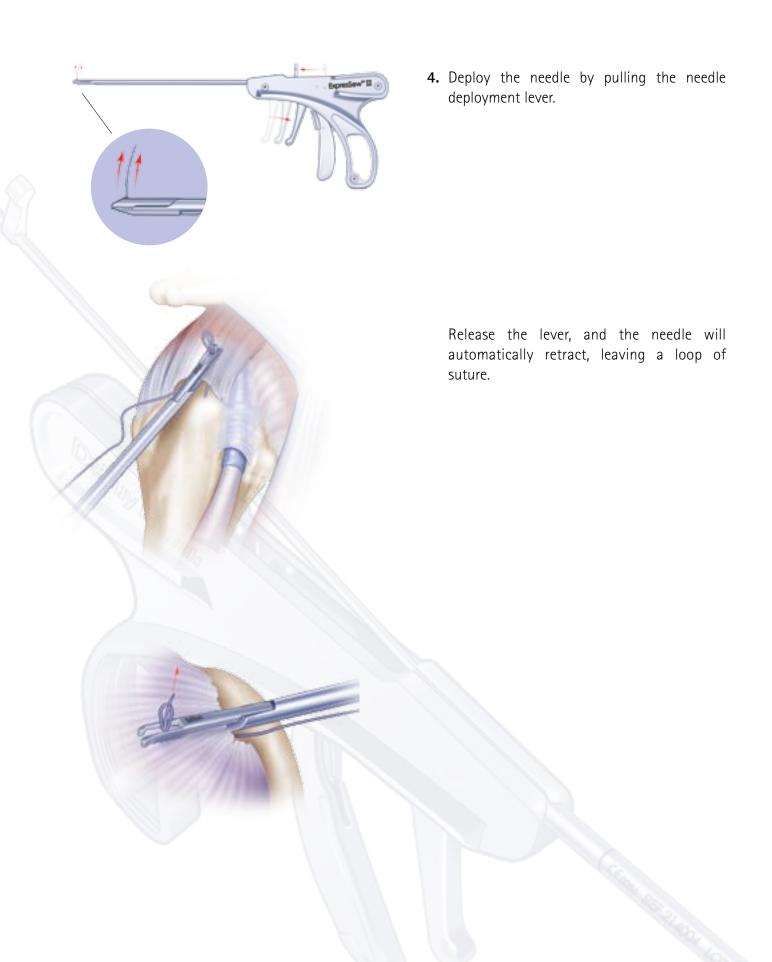

EXPRESSEW II FLEXIBLE SUTURE PASSER



5. Reach on top of the Rotator Cuff and grasp the suture in the jaws of the EXPRESSEW II.

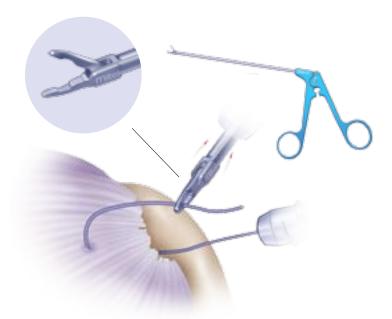

Alternative technique: Grasp the suture from an accessory portal with the DePuy Mitek Grasper-Grabber.



EXPRESSEW II			
Catalog #	Description	Quantity Order Qty	
214004	EXPRESSEW II Flexible Suture Passer	1/box	
214005	EXPRESSEW II Needles	5/box	
270120	Grasper-Grabber	1/box	

EXPRESSEW II			
Catalog #	Description	Quantity Order Qty	
214004	EXPRESSEW II Flexible Suture Passer	1/box	
214005	EXPRESSEW II Needles	5/box	
270120	Grasper-Grabber	1/box	

VERSALOK™ Anchors and Instruments				
Catalog #	Description	Quantity Order Qty		
210808	VERSALOK Anchor with ORTHOCORD™	1/box		
214710	Deployment Gun	1/box		
214711	2.9mm Awl	1/box		
215440	VERSALOK Caddy	1/box		

FASTIN	® RC Threaded Anchors (non-needle codes)	
Catalog #	Description	Quantity Order Qty
222723	5mm FASTIN RC Anchor with ETHIBOND®	1/box
222764	6.5mm FASTIN RC Anchor with ETHIBOND	1/box
222743	5mm FASTIN RC Anchor with PANACRYL®	1/box
222765	6.5mm FASTIN RC Anchor with PANACRYL	1/box
222993	5mm FASTIN RC Anchor with ORTHOCORD	1/box
222994	6.5mm FASTIN RC Anchor with ORTHOCORD	1/box

SPIRALOK™ Anchors and Instruments (non-needle codes)					
Catalog #	Description	Quantity Order Qty			
222986	SPIRALOK 5.0mm ORTHOCORD Violet and Blue	1/box			
222988	SPIRALOK 6.5mm ORTHOCORD Violet and Blue	1/box			
222964	SPIRALOK 5.0mm ETHIBOND	1/box			
222965	SPIRALOK 5.0mm PANACRYL	1/box			
222966	SPIRALOK 6.5mm ETHIBOND	1/box			
222967	SPIRALOK 6.5mm PANACRYL	1/box			
222972	SPIRALOK AWI	1/box			
222975	SPIRALOK Tap	1/box			
270100	SPIRALOK Awi/Tap Combo	1/box			











