Dyonics® Power System Powered Instrumentation

Operations Manual

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Manuale operativo

Driftshandleiding

Bedieningshandleiding

Symbols

	On
	Off
	Slower
	Faster
	Motor Drive Unit
	Footswitch
	Minimum
	Maximum
	Speed
	See Instructions for Use
	Type B Equipment
	Forward
	Reverse

TABLE OF CONTENTS

PREFACE

This manual provides the information you need to operate and maintain the Dyonics *Power* Endoscopic Powered Instrumentation. It is essential that you read and understand all the information in this manual before using or maintaining the equipment.

INDICATIONS

Indications 2

WARNINGS AND CAUTIONS

Warnings 3
 Cautions 4

INTRODUCTION

Introduction 5

SYSTEM COMPONENTS

Handpieces 5
 Sagittal Saw Blades
 Drill Bits, K-Wires 6
 Foot Control 6
 Cutting Technique 7

SET UP

Set Up 7

OPERATION

Pistol Grip Drill Handpiece . . . 8
 K-Wire or Steinmann Pin . . . 8
 Jacobs Chucks 9
 Removing the Chuck 9
 Sagittal Saw Module 10
 Pencil Grip Sagittal
 Saw Handpiece 10

CLEANING & STERILIZATION

Cleaning 11
 Disassembly 11
 Cleaning 11
 Sterilization Parameters . . . 11
 Accidental Immersion . . . 12

TROUBLESHOOTING

Troubleshooting 12

SHIPPING & ENVIRONMENTAL PARAMETERS

Conditions 13

WARRANTY

New Product Warranty. Back Cover
 Service Replacement
 Units Warranty Back Cover

Figures

Dyonics *Power* Control Unit
 Front Panel Cover
 Drill Handpiece 5
 Saw Handpiece 5
 Insert Wire/Pin Into Drill. . . . 8
 Using Jacobs Chucks 9
 Removing Jacobs Chuck 9
 Sagittal Saw Module with Drill . 10
 Sagittal Saw Handpiece . . . 10

INDICATIONS

The Dyonics drills, wire/pin drivers, sagittal saw, and other accessories, when used in conjunction with the Dyonics *Power* Control Unit (REF 7205841), are indicated for drilling and cutting bone.

WARNINGS

Prior to use, this Operations Manual should be read and understood by all operating room staff handling the equipment. As with any surgical instrument, there are important health and safety considerations. These are listed below and highlighted within the text.

Note: All personnel should become familiar with the powered instruments before they are set up for use in any procedure. Personnel in-serviced should include, but not be limited to central processing personnel, members of the surgical team, and bio-engineering department.

WARNINGS

PRIOR TO USE, ALL SYSTEM COMPONENTS SHOULD BE INSPECTED AND OPERATED TO DETERMINE IF THERE IS PRODUCT DAMAGE AND/OR MALFUNCTION. DO NOT USE IF PRODUCT IS DAMAGED AND/OR MALFUNCTIONING; RETURN IMMEDIATELY AFTER CONTACTING SMITH & NEPHEW, INC. CUSTOMER SUPPORT: 1-800-343-5717.

CHECK SAGITTAL SAW BLADES AND DRILLS. ENSURE THAT SAGITTAL SAW BLADES, WIRES, PINS, AND DRILLS, ARE NOT DULL OR BENT, AND THAT THEY LOCK CORRECTLY INTO THE HANDPIECE.

INSPECT AND TEST THE HANDPIECES:

- **INSPECT THE HANDPIECES FOR DAMAGE, CORROSION, OR EXCESSIVE WEAR.**
- **ENSURE THAT WHEN THE SAFETY LOCK OF HANDPIECES IS ON "SAFE," IT PREVENTS ACTIVATION OF THE MOTOR. ENSURE THAT THE ACTIVATION TRIGGER AND THROTTLE LEVER DO NOT STICK IN THE FULL "RUN" POSITION.**

PRIOR TO INSTALLING OR REMOVING SAGITTAL SAW BLADES, DRILLS, OR ANY OTHER ATTACHMENTS, ENSURE THAT THE HANDPIECE IS IN "SAFE" MODE.

THE SAFETY LOCK SHOULD BE IN THE "SAFE" POSITION WHEN EITHER HANDPIECE IS NOT IN USE OR WHEN INSERTING OR REMOVING MODULES, BLADES, DRILLS OR ANY OTHER ATTACHMENTS.

THE SAGITTAL SAW AND THE PISTOL GRIP DRILL WILL BE ACTIVATED VIA THE FOOT-SWITCH EVEN WHEN IN THE "SAFE" POSITION.

THE TRIGGER/THROTTLE LEVER IS ACTIVE AS THE HANDPIECE CABLE IS BEING PLUGGED INTO EITHER THE HANDPIECE OR THE CONTROL UNIT. KEEP HANDPIECES IN THE "SAFE" MODE TO AVOID INADVERTENT ACTIVATION. FOOTSWITCHES WILL ACTIVATE THE DRILL AND SAGITTAL SAW EVEN WHEN THEY ARE IN THE "SAFE" POSITION.

AVOID USING A SAGITTAL SAW BLADE AS A LEVER. EXCESSIVE FORCE MAY LEAD TO FRACTURE OF THE SAW BLADE.

IF USING A FOOTSWITCH, ENSURE THAT THE FOOTSWITCH IS NOT ACCIDENTALLY ACTIVATED WHILE INSERTING THE BLADE.

SAGITTAL SAW BLADES ARE SINGLE USE, DISPOSABLE DEVICES. DO NOT REUSE.

CAUTIONS

CAUTIONS

U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

Standard operating conditions eventually may cause overheating or total failure of the instrument. With the blade or drill bit attached, test run the handpiece in the sterile field for three 10-second intervals, checking for any indication of irregular speed, noise, excessive heat, or vibration. Irregular grinding noises may indicate impending failure or overheating of the handpiece.

Periodically monitor the temperature of the nose section. The temperature should not rise above 115° F and should not become uncomfortable to touch with gloved fingers.

Only Smith & Nephew sagittal saw blades are compatible with the Sagittal Saw Module or Handpiece.

Dull saw blades are major contributors to problems with powered surgical instruments. They require the surgeon to exert greater force, which increases stress, heat, and wear on gears, bearings, and other mechanisms. Use of excessive force also increases the risk of thermal necrosis and overheating of handpieces.

It is difficult to determine when a drill bit is dull. Simply counting the number of uses is not adequate. While certain procedures cause relatively little wear on a drill bit, other procedures, especially where the drill scrapes against metal templates or retractors, produce immediate wear or damage to the drill bit.

Ensure that cable connector pins are dry at both ends prior to use.

DO NOT immerse the handpieces in saline, disinfectant, or any other liquid.

DO NOT use an ultrasonic cleaner or any other automated/sterilizing cleaning equipment. Ultrasonic cleaning can damage the bearings in the handpieces, potentially resulting in overheating or seizure.

DO NOT lubricate or oil the handpieces. Lubrication will clog the motor and prevent it from starting. Also take special precautions to avoid the use of cleaners which contain lubricants.

Soaking the Handpiece cable may cause damage or corrosion to the connector pins inside, the control unit, and/or the handpiece itself.

The handpiece cable should be sterilized in the same manner and location as the Pistol Grip Drill and Sagittal Saw.

DO NOT run instruments while warm. Cool by exposure to room temperature.

DO NOT immerse handpiece(s) in liquid to cool.

DO NOT process powered surgical instruments in a washer/sterilizer. The washer cycle will damage the motor and other internal components.

Electrical safety testing should be performed by a biomedical engineer or other qualified person.

This equipment is designed and tested to minimize interference with other electrical equipment. However, if interference occurs with other equipment, it may be corrected by one or more of the following measures:

- Reorient or relocate this equipment, the other equipment, or both.
- Increase the separation between the pieces of equipment.
- Connect the pieces of equipment into different outlets or circuits.
- Consult a biomedical engineer.

This equipment contains electronic printed circuit assemblies. At the end of the useful life of the equipment it should be disposed of in accordance with any applicable national or institutional related policy relating to obsolete electronic equipment.

INTRODUCTION/SYSTEM COMPONENTS

INTRODUCTION

The following Powered Instruments can be used with the Dyonics Power Control Unit (REF 7205841):

- ◇ Pistol Grip Drill Handpiece REF 7205785
 - Jacobs Chuck—5/32" REF 7205792
 - Jacobs Chuck—1/4" REF 7205794
 - Keyless Jacobs Chuck—1/8" REF 7205793
 - Keyless Jacobs Chuck—1/4" REF 7205795
 - Sagittal Saw Module REF 7205791
 - Wire Driver REF 7205789
 - Pin Driver REF 7205790
- ◇ Cable Assembly REF 7205788
- ◇ Sterilization Tray REF 7205787
- ◇ Inline Sagittal Saw Handpiece REF 7205786

◇ Saw Blades—STERILE, with Markings:

W (mm)	x (mm)	L (mm)	x (mm)	T (mm)	Depth Stop	Titanium Nitride Finish	REF No.
4.5	25.5	0.4	NO	NO	NO	NO	7205806
5.8	25.4	0.4	NO	NO	NO	NO	7205803
9.4	25.5	0.4	NO	NO	NO	NO	7205805
9.4	25.5	0.4	NO	NO	NO	NO	7205804*
19.1	25.4	0.4	NO	NO	NO	NO	7205802
5.8	25.4	0.4	NO	YES	NO	YES	7205896
9.4	25.5	0.4	NO	YES	NO	YES	7205897*
19.1	25.4	0.4	NO	YES	NO	YES	7205895

*Fine Teeth



◇ Saw Blades—STERILE, NO Markings:

W (mm)	x (mm)	L (mm)	x (mm)	T (mm)	Depth Stop	Titanium Nitride Finish	REF No.
9.4	10.0	0.4	10 mm	YES	NO	YES	7205807
9.4	12.0	0.4	12 mm	YES	NO	YES	7205808
9.4	15.0	0.4	15 mm	YES	NO	YES	7205809
9.4	10.0	0.4	10 mm	NO	NO	NO	7205898
9.4	12.0	0.4	12 mm	NO	NO	NO	7205899
9.4	15.0	0.4	15 mm	NO	NO	NO	7205900

SYSTEM COMPONENTS

HANDPIECES (DRILL, REF 7205785 OR SAGITTAL SAW, REF 7205786)

The Inline Sagittal Saw is activated by pressing a *throttle lever* and the Pistol Grip Drill is activated by pressing a *trigger*. These are variable speed devices and can also be activated via a footswitch. The *throttle lever* and *trigger* have two positions: "SAFE" and "RUN." When in the "SAFE" position, the *throttle lever* or *trigger* is inactive.

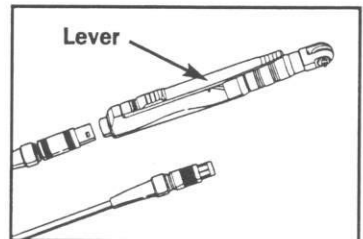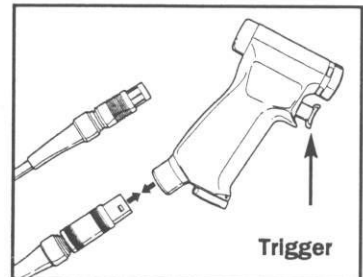! WARNING: THE SAFETY LOCK SHOULD BE IN THE "SAFE" POSITION WHEN EITHER HANDPIECE IS NOT IN USE OR WHEN INSERTING OR REMOVING MODULES, BLADES, DRILLS OR ANY OTHER ATTACHMENTS.

! WARNING: THE SAGITTAL SAW AND THE PISTOL GRIP DRILL WILL BE ACTIVATED VIA THE FOOTSWITCH EVEN WHEN IN THE "SAFE" POSITION.

SYSTEM COMPONENTS

SAGITTAL SAW BLADES, DRILL BITS, AND K-WIRES

The Pistol Grip Handpiece will accept a Sagittal Saw Module, a variety of drill chucks, a Pin Driver, or a K-Wire Driver. The Inline Sagittal Saw Handpiece will accept a variety of saw blades. All carbide or stainless steel saw blades and drills have sharp, precision ground teeth or flutes designed to provide optimum cutting performance.

Note: Use a new sagittal saw blade or drill for each procedure. Using disposable accessories reduces handpiece wear, improves surgical cutting, and ensures quality patient care.

Caution: Dull saw blades are major contributors to problems with powered surgical instruments. They require the surgeon to exert greater force, which increases stress, heat, and wear on gears, bearings, and other mechanisms. Use of excessive force also increases the risk of thermal necrosis and overheating of handpieces.

Caution: Sagittal saw blades are provided as sterile, single use products. Do not repackage or reuse sagittal saw blades.

Caution: Only Smith & Nephew sagittal saw blades are compatible with the Sagittal Saw Module or Handpiece.

Caution: It is difficult to determine when a drill bit is dull. Simply counting the number of uses is not adequate. While certain procedures cause relatively little wear on a drill bit, other procedures, especially where the drill scrapes against metal templates or retractors, produce immediate wear or damage to the drill bit.

FOOT CONTROL

When the footswitch is plugged into the Dyonics Power Control Unit the green LED light (lower left corner of the control unit) is illuminated and active. All standard Dyonics footswitches will operate the Drill, Sagittal Saw Module, or Sagittal Saw. The sagittal saw (and module) runs in one direction when activated. The drill runs in either the forward or reverse mode.

Note: If the trigger/throttle lever is active and then the footswitch is depressed, the footswitch will NOT activate. BOTH lever AND footswitch must be released for one second, and then only one depressed, to activate the handpiece.

Note: The oscillate function from the footswitch is inoperable for the drill and sagittal saw.

 **WARNING: THE TRIGGER/ THROTTLE LEVER IS ACTIVE AS THE HANDPIECE CABLE IS BEING PLUGGED INTO EITHER THE HANDPIECE OR THE CONTROL UNIT. KEEP HANDPIECES IN THE "SAFE" MODE TO AVOID INADVERTENT ACTIVATION. FOOTSWITCHES WILL ACTIVATE THE DRILL AND SAGITTAL SAW EVEN WHEN THEY ARE IN THE "SAFE" POSITION.**

SYSTEM COMPONENTS/SETUP AND OPERATION

CUTTING TECHNIQUE

 **WARNING: AVOID USING A SAGITTAL SAW BLADE AS A LEVER. EXCESSIVE FORCE MAY LEAD TO FRACTURE OF THE SAW BLADE.**

Caution—Take the following precautions to avoid damaging the handpiece:

- *Avoid excessive pressure on the sagittal saw or saw module when cutting. Use a light “pencil grip” and let the instrument do the work. Heavy force will rapidly wear components.*
- *Avoid plunging and “burying” a sagittal saw blade straight into bone. This will lock the blade and prevent it from completing its stroke, damaging or possibly breaking the oscillating mechanism. Use a slight “back and forth” or “side to side” motion when cutting. This will keep the cut wide enough to permit the blade to complete its stroke and maintain speed. Continuous stalling of the motor will cause rapid heating of the handpieces.*
- *Take special care when making curved cuts to avoid locking the blade. Always keep the blade free to complete its stroke.*
- *The sagittal saw blade may flex or whip at the tip when running at full speed in free air. This occurs due to the thinness of the blade which is also a key to its fast cutting ability. Always begin the cut with the throttle set for moderate speed. After establishing the cut in the bone, advance the throttle to full speed. The blade will not flex once in the bone.*

SETUP AND OPERATION

SETTING-UP THE DYONICS POWERED INSTRUMENTS FOR USE IN SURGICAL PROCEDURES

1. Place the Control Unit on a cart, turn power switch on front panel to the OFF position, plug the unit's power cord into the rear panel power connector and the other end into a grounded AC power source (wall).
Note: “OFF” position is designated by “O” and the “ON” position “I.”
2. Push the Power switch to the On “I” position.
3. Connect the powered instrument handpiece cable (REF 7205788) into either **Handpiece A** or **B** receptacle on the Front Panel of the Control Unit. Select the **A** or **B** channel by pressing the **A** or **B** button on the Front Panel.

Caution: Ensure that cable connector pins are dry at both ends prior to use.

Note: Be sure to align the pins in the cable with the holes in the Control Unit receptacle before inserting.

4. Firmly hold the handpiece and insert the instrument cable into the handpiece receptacle.
Note: Avoid pressing the trigger, throttle lever or footswitch while attaching the instrument cable. If this occurs, the handpiece will not run until trigger or throttle lever is released for one second and then activated.
5. If using a footswitch, connect it to the appropriate receptacle on the left side of the control unit front panel. Ensure that pins in cable are aligned with the holes in the receptacle before inserting.


SETUP AND OPERATION

DRILL HANDPIECE (REF 7205785)


The Pistol Grip Drill is a modular device with attachments for a wire driver, pin driver, sagittal saw module, and a variety of drill chucks. The system includes a handpiece, an automatic wire collet for K-wires up to .062" (1.6 mm), an automatic pin collect which accepts pins up to .125" (3.2 mm), and a selection of Jacobs chucks.

To insert a K-wire or Steinmann pin into the Pistol Grip Drill:

1. Attach handpiece cable from control unit to drill handpiece receptacle and set safety lock to ON by rotating the trigger button to either side.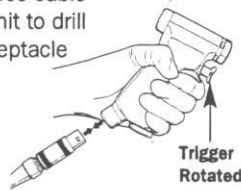


2. Snap wire or pin collet into the nose of the drill.

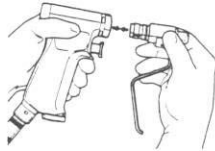3. Screw the wire guard into the back of the handpiece to protect the operator from the point of the wire or pin.

4. Insert wire or pin into the front or back of the handpiece.

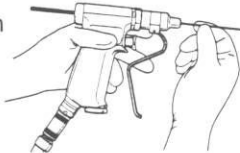
5. Select Forward ("FWD") or Reverse ("REV") on the bottom of the handpiece.

6. Move the trigger button out of the "SAFE" position by rotating it straight up and down.

7. Squeeze the wire/pin advance lever and hold it down.

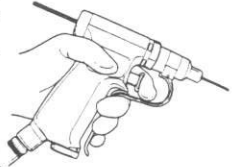8. Depress the trigger to drive the wire/pin. The trigger is pressure sensitive providing for variable speed operation.

9. To obtain additional wire or pin length, release the wire/pin advance lever and trigger then pull back on the instrument. Then squeeze the wire/pin advance lever and depress trigger to drive the wire.

10. To remove threaded wires or pins, put the drill in "Reverse," squeeze the wire/pin advance lever and then depress the trigger.

11. To release the wire or pin collet, put the trigger in the "SAFE" position.

12. Depress the release button on the top of the handpiece and pull the collet out.

SETUP AND OPERATION

To use the Jacobs Chucks:

1. Set safety lock to "ON" position by rotating the trigger button to either side.

! WARNING: IF USING A FOOTSWITCH, ENSURE THAT THE FOOTSWITCH IS NOT ACCIDENTALLY ACTIVATED WHILE INSERTING THE BLADE.

2. Snap Jacobs keyed or keyless chuck into nose of drill.

3. Open keyed chuck (via key)


or

keyless chuck (by rotating rings in opposite direction).

4. Insert drill bit or wire/pin into nose of chuck.

5. Use Jacobs chuck key to close drill bit or wire/pin into place,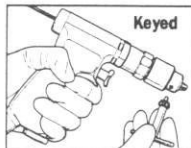

or

if using keyless chuck, rotate rings in opposite direction.

6. Select Forward ("FWD") or Reverse ("REV") on the bottom of the drill.

7. Move the trigger button out of the "SAFE" position by rotating it straight up and down.

8. Depress the trigger.


To remove the chuck:

1. Place handpiece trigger in "SAFE" position.

2. Remove drill bit, wire or pin.

3. Ensure trigger is in "SAFE" position, then depress release button on top of drill and pull out chuck.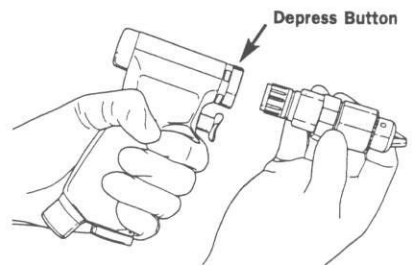

SETUP AND OPERATION

To use the Sagittal Saw Module (REF 7205791) with the Pistol Grip Drill:

1. Set safety lock on the drill to "ON" by rotating the trigger button to either side.


WARNING: IF USING A FOOTSWITCH, ENSURE THAT THE FOOTSWITCH IS NOT ACCIDENTALLY ACTIVATED WHILE INSERTING THE BLADE.

2. Snap Sagittal Saw Module into nose of drill.

3. Press pin on the side of the module to open blade retention jaws, insert saw blade making sure that the blade is fully seated, then release pin to secure the blade.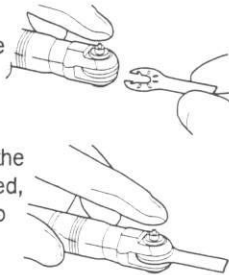

4. Select Forward ("FWD") or Reverse ("REV") on the bottom of the handpiece.
- 
5. Move trigger button from the "SAFE" position by rotating it straight up and down.
 6. Depress the trigger to activate the handpiece.
 7. To remove the blade, move trigger to the "Safe" position, then press pin to release jaws and remove blade.
 8. To remove the Sagittal Saw Module from the drill, ensure trigger is in the "SAFE" position, then depress release button on top of handpiece and remove the Sagittal Saw Module.

PENCIL GRIP SAGITTAL SAW HANDPIECE (REF 7205786)


To Insert a blade Into the Pencil Grip Sagittal Saw:

1. Move the safety lock on the throttle lever to "SAFE".

WARNING: IF USING A FOOTSWITCH, ENSURE THAT THE FOOTSWITCH IS NOT ACCIDENTALLY ACTIVATED WHILE INSERTING THE BLADE.

2. Press pin on the side of the module to open blade retention jaws; insert saw blade making sure that the blade is fully seated, then release pin to secure the blade.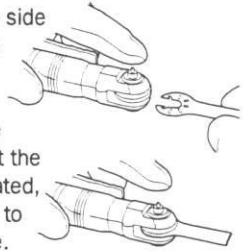

Note: The blade may be located in one of five positions.

3. Move the safety lock out of the "SAFE" position.
4. Depress the throttle lever to activate the instrument.

To remove a blade from the Pencil Grip Sagittal Saw:

1. To remove the blade, return lever to the "SAFE" position.

2. Press pin to release retention jaws and remove blade.

Caution: Standard operating conditions eventually may cause overheating or total failure of the instrument. Periodically monitor the temperature of the nose section. The temperature should not rise above 115° F and should not become uncomfortable to touch with gloved fingers.

CLEANING AND STERILIZATION

Dried blood, saline, and other deposits inside the handpieces are a major cause of equipment malfunction.

Caution: DO NOT immerse the handpieces in saline, disinfectant, or any other liquid.

Caution: DO NOT use an ultrasonic cleaner or any other automated/sterilizing cleaning equipment. Ultrasonic cleaning can damage the bearings in the handpieces, potentially resulting in overheating or seizure.

Caution: DO NOT lubricate or oil the handpieces. Lubrication will clog the motor and prevent it from starting. Also take special precautions to avoid the use of cleaners which contain lubricants.

CLEANING

1. Disassemble and transport to the decontamination area.
 - a. If using the Sagittal Saw, slide the safety lock to the "SAFE" position. If using the Drill, rotate the trigger to the "SAFE" position. Unplug the footswitch cable from the control unit.
 - b. Turn the power on the control unit to "OFF;" unplug the power cord from the wall outlet.
 - c. Disconnect all handpiece cables from the control unit (not from the handpiece).
 - d. Remove and discard used saw blades and drills after use, handling them as any contaminated sharp instrument.

 **WARNING: SAGITTAL SAW BLADES ARE SINGLE USE, DISPOSABLE DEVICES. DO NOT REUSE.**

2. Clean the device immediately, using an appropriate detergent solution.

Caution: Soaking the Handpiece cable may cause damage or corrosion to the connector pins inside, the control unit, and/or the handpiece itself.

- a. Ensure that the handpiece is connected to the handpiece cable to keep detergent solution and water out of the motor.
 - b. Clean the handpiece cable thoroughly with warm water, mild detergent, and a soft brush. Pay vigorous attention to crevices.
3. Use a cannulation brush on cannulated components (i.e., drills, wire and pin drivers).
 - a. Clean the cannulated shaft with a small cannulation brush.
 - b. Clean inside drill and wire collets.
 4. Flush under pressure the noses of drills and wire drivers to remove blood, debris, and saline deposits.
 5. Rinse, with the handpiece cable connected, components under running water to remove all traces of detergent solution. If possible, use distilled water for the final rinse.
 6. Dry and disconnect the handpiece cable from the handpiece, grasping it firmly on the grooved or knurled area of the plug. Be sure to pull straight back and DO NOT pull on the strain relief immediately behind it.

Note: The handpiece cable has a mechanical snap-lock mechanism. To disconnect, pull on the grooved or knurled area of the plug (not on the strain relief immediately behind it).

STERILIZATION

Caution: The handpiece cable should be sterilized in the same manner and location as the Pistol Grip Drill and Sagittal Saw.

1. **Sterilization parameters:** Smith & Nephew's powered surgical instruments (including handpieces, handpiece cable, and reusable accessories), are sterilized by steam, using either a gravity displacement or pre-vacuum sterilization ONLY.

CLEANING AND STERILIZATION/TROUBLESHOOTING

First Choice: Pre-vacuum steam sterilization for 4 minutes at 270° F (132° C), 8 minutes drying time.

Second Choice: Gravity displacement steam sterilization for 35 minutes at 270° F (132° C), 8 minutes drying time.

Note: These parameters have been validated to ensure sterility. Sterilizer functioning should be monitored at regular intervals with biological indicators to ensure products have been subjected to sterilization conditions.

Caution: DO NOT run instruments while warm. Cool by exposure to room temperature.

Caution: DO NOT immerse handpiece(s) in liquid to cool.

Caution: DO NOT process powered surgical instruments in a washer/sterilizer. The washer cycle will damage the motor and other internal components.

ELECTRICAL INTERFERENCE

Caution: This equipment is designed and tested to minimize interference with other electrical equipment. However, if interference occurs with other equipment, it may be corrected by one or more of the following measures:

- Reorient or relocate this equipment, the other equipment, or both.
- Increase the separation between the pieces of equipment.
- Connect the pieces of equipment into different outlets or circuits.
- Consult a biomedical engineer.

PREVENTATIVE MAINTENANCE

Caution: Electrical safety testing should be performed by a biomedical engineer or other qualified person.

2. Accidental Immersion:

If a handpiece is accidentally immersed in saline, disinfectant, cleaning fluid, or any other corrosive substance, take the following steps:

- a. Totally immerse the handpiece in distilled water for one minute to dilute the corrosive fluid. DO NOT allow the water to dry in the handpiece.
- b. Immediately after soaking, steam sterilize in a pre-vacuum sterilizer at 270° F (132° C) for 4 minutes followed by an 8 minute drying time. Sterilizing will dry out the handpiece, avoid rusting and prevent contamination from collecting in the motor.

TROUBLESHOOTING

1. Handpiece cable (REF 7205788) is difficult to insert into handpiece or control unit.

- a. Ensure that the pins on the handpiece cable are aligned with the matching holes or RED DOT's on the Control Unit and/or handpiece receptacle. This connection is a tight fit to keep particles from moving inside the handpiece.
- b. Ensure that each end of the plug is fully engaged via the "snap lock."

Note: The termination connectors should retract into the plug while pulling back on the notched portion of the plug sleeve. If the connector sleeves do not retract, send the handpiece cable to the factory for inspection, and use an alternate handpiece cable.

TROUBLESHOOTING/SHIPPING AND ENVIRONMENTAL

2. Handpiece will not start.

- a. Ensure that the Control Unit is "ON" (the main power switch is in the "I" position) and the display panel and power switch are illuminated.
- b. Ensure that either the throttle lever or trigger is in the "RUN" not "SAFE" position.
- c. If using the footswitch, ensure that the connector is fully inserted into the receptacle.
- d. Test handpiece in the other handpiece receptacle (A or B).
- e. Test the handpiece with another handpiece cable. If new handpiece cable starts the handpiece, return first handpiece cable for repair.
- f. Test a different handpiece in both handpiece receptacles (A and B). If the new handpiece runs properly, return the faulty handpiece for repair.
- g. If using a footswitch, disconnect it and plug handpiece into the Control Unit. If the handpiece runs properly, return the footswitch for repair.
- h. If the above steps do not cause the handpiece to run properly, return the Dyonics Power system for repair.

Note: Pressing either the trigger or throttle lever too slowly may prevent the handpiece from starting.

3. Handpiece runs slowly.

- a. Ensure that the safety lock is in the "RUN" position.
- b. Repeat steps "b-h" above under "Handpiece will not start."

- c. Pressing the throttle lever or trigger to slowly may prevent handpiece from achieving the maximum set speed. Release trigger or lever and press again after one second.

4. Motor overheats.

- a. Turn the handpiece off and let it cool down. When the handpiece has cooled, check to see that the blade is properly seated and locked.
- b. If the handpiece overheats again, send it to Smith & Nephew for repair.

5. Trigger or Throttle Lever sticks.

- a. Repeat cleaning and sterilization procedures.
- b. If problem persists, return product to Smith & Nephew.

SHIPPING AND ENVIRONMENTAL PARAMETERS

This device has been tested and proven to function after repeated exposure to the following conditions:

- a. Temperature: 0° F (-17.8° C) through 120° F (48.8° C)
- b. Humidity: 0% through 90%
- c. Pressure: 645 mmHg through 795 mmHg

ENVIRONMENTAL PROTECTION

Caution: This equipment contains electronic printed circuit assemblies. At the end of the useful life of the equipment it should be disposed of in accordance with any applicable national or institutional related policy relating to obsolete electronic equipment.

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