

®

stryker



# X7000

Light Source Operating and Maintenance Manual

stryker®  
ENDOSCOPY



# Table of Contents

<b>Table of Contents</b> . . . . .	<b>1</b>
<b>Warnings and Cautions</b> . . . . .	<b>3</b>
<b>Product Description / Intended Use</b> . . . . .	<b>5</b>
<b>Setting Up the X7000</b> . . . . .	<b>8</b>
Connecting the AC Power Cable . . . . .	<b>8</b>
Connecting the Camera and Monitor . . . . .	<b>9</b>
Connecting the Light Cable . . . . .	<b>10</b>
<b>Operating the X7000</b> . . . . .	<b>12</b>
Powering the System On and Off . . . . .	<b>12</b>
Selecting the Operation Mode . . . . .	<b>13</b>
Adjusting the Brightness . . . . .	<b>13</b>
Receiving Feedback from the X7000 . . . . .	<b>14</b>
Safety Shutoff . . . . .	<b>15</b>
Checking the ESST Feature . . . . .	<b>16</b>
Using the X7000 with a Voice-Controlled System Interface . . . . .	<b>16</b>
Using the Ethernet Interface . . . . .	<b>17</b>
<b>Cleaning and Maintenance</b> . . . . .	<b>18</b>
Cleaning the X7000 . . . . .	<b>18</b>
Caring for the Lamp Module . . . . .	<b>18</b>
Replacing the Lamp Module . . . . .	<b>19</b>
Replacing the Fuses . . . . .	<b>21</b>
Disposing of the X7000 . . . . .	<b>21</b>
<b>Troubleshooting</b> . . . . .	<b>22</b>
<b>Technical Specifications</b> . . . . .	<b>25</b>
<b>Electromagnetic Compatibility Tables</b> . . . . .	<b>27</b>
<b>Warranty</b> . . . . .	<b>31</b>
<b>Service and Claims</b> . . . . .	<b>32</b>

**Other Service . . . . . 33**

## Warnings and Cautions

Please read this manual and follow its instructions carefully. The words **warning**, **caution**, and **note** carry special meanings and should be carefully reviewed:

**Warning** The personal safety of the patient or physician may be involved. Disregarding this information could result in injury to the patient or physician.

**Caution** Special service procedures or precautions must be followed to avoid damaging the instrument.

**Note** Special information to make maintenance easier or important information more clear.



An exclamation mark within a triangle is intended to alert the user to the presence of important operating and maintenance instructions in the literature accompanying the product.



A lightning bolt within a triangle is intended to warn of the presence of hazardous voltage. Refer all service to authorized personnel.

To avoid potential serious injury to the user and the patient and/or damage to this device, the user must:

1. Read this operating manual thoroughly and be familiar with its contents prior to using this equipment.
2. Carefully unpack the unit and check if any damage occurred during shipment. If damage is detected, please refer to the *Service and Claims* section in this manual.
3. Be a qualified physician, having complete knowledge of the use of this equipment.
4. Test this equipment prior to a surgical procedure. This unit was fully tested at the factory before shipment.
5. Attempt no internal repairs or adjustments not specifically detailed in this operating manual.
6. Never sterilize any part of the X7000 console.
7. Disconnect the X7000 from the electrical outlet when inspecting the fuses.

The warranty is void if any of these warnings is disregarded.

Stryker Endoscopy accepts full responsibility for the effects on safety, reliability, and performance of the equipment only if:

- Readjustments, modifications, and/or repairs are carried out exclusively by Stryker Endoscopy.
- The electrical installation of the relevant operating room complies with the applicable IEC, CEC, and NEC requirements.

Stryker Endoscopy reserves the right to make improvements in the product(s) described herein. Product(s), therefore, may not agree in detail with the published design or specifications. All specifications are subject to change without notice. Please contact the local Stryker Endoscopy Distributor listed in the *Other Service* section, or phone your local Stryker Endoscopy sales representative or agent for information on changes and new products.

**Warning** Federal law (United States of America) restricts this device to use by, or on order of, a physician.



#### Other Symbol Definitions:



Type CF Applied Part



Protective Earth Ground



Equipotentiality



Denotes compliance to CSA 22.2 No.601.1-M90 and UL2601-1.

## Product Description / Intended Use

The Stryker Endoscopy X7000 Xenon Light Source is a light-generating unit designed to illuminate surgical sites during endoscopic applications. The X7000 uses a 300-watt xenon lamp to generate bright, crisp light, which it delivers to the surgical site via a fiberoptic light cable. The X7000 is compatible with all Stryker light cables, and, with the proper light cable and adapters, can connect to any flexible or rigid endoscope.

The X7000 is equipped with Electronic Scope Sensing Technology (ESST), a special safety feature that helps prevent accidental burns caused by an unattended light cable. When operated with an ESST light cable, the X7000 senses when the scope and the light cable are separated and places the light source in STANDBY mode. In STANDBY mode, the X7000 will reduce light output to a minimum, preventing the light cable from generating excessive heat.

The Stryker Endoscopy X7000 Xenon Light Source consists of one of each of the following:

- light source console
- power cord
- video cable (purchased separately)
- xenon lamp module (spare or replacement lamp modules can be purchased separately)

The features of the X7000 console are described in Figure 1 on the following page.

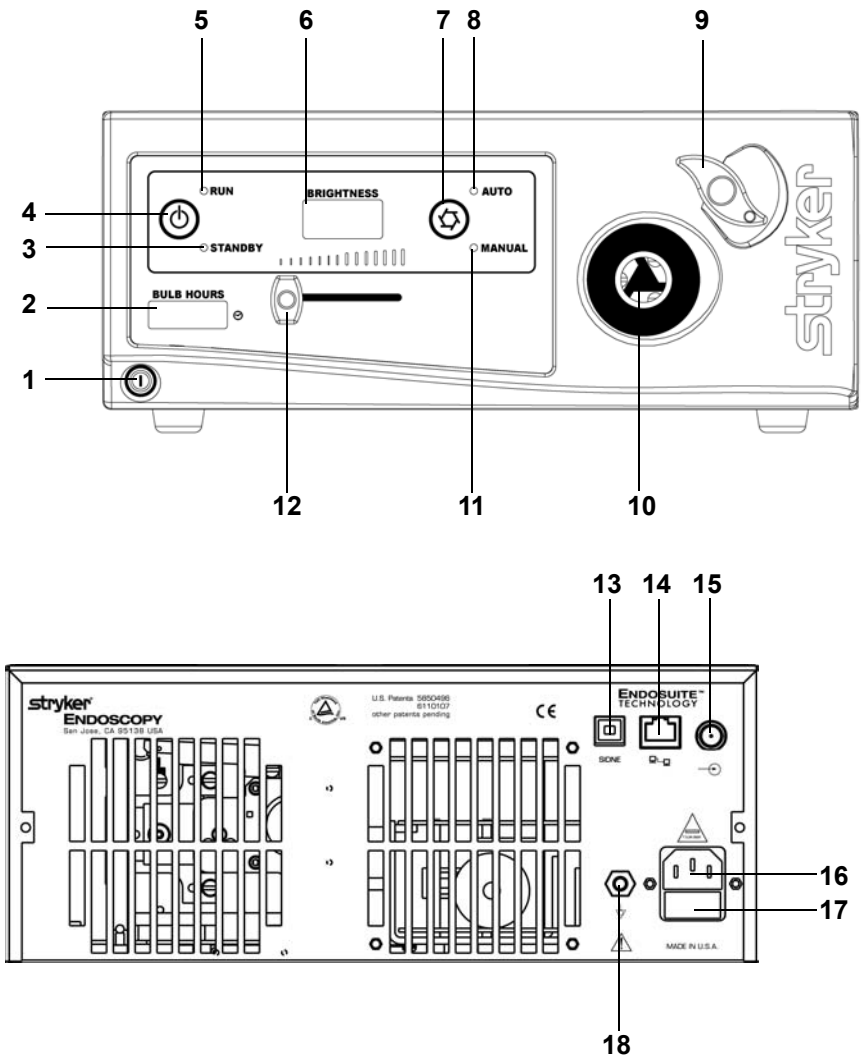


Figure 1: The X7000 console, front panel (top) and rear panel (bottom)



1. **Power Button:** Powers the unit on and off.
2. **Bulb-Hours LCD:** Displays the total number of hours the lamp has operated since the last lamp change.
3. **STANDBY LED:** Illuminates when the unit is operating in the STANDBY mode. In STANDBY mode, light output is reduced to a minimum level.
4. **Mode Switch:** Selects either STANDBY or RUN mode when pressed. STANDBY is the default mode when the light source is powered on.
5. **RUN LED:** Illuminates when the unit is operating in the RUN mode. In RUN mode, light output is determined by the automatic or manual shutter.
6. **Brightness LCD:** Indicates the light intensity level (0-100%).
7. **Shutter Switch:** Selects either the AUTO or MANUAL shutter mode when pressed. MANUAL is the default mode when the light source is powered on.
8. **AUTO LED:** Illuminates when the shutter is in the AUTO mode. In AUTO mode, light output is determined by the brightness setting and the input video signal.
9. **Jaw Handle:** Opens the fiberoptic-cable holder.
10. **Cable Clamp:** Grasps the light-source end of an inserted fiberoptic cable. (When no cable is inserted, the lamp automatically turns off to save bulb hours.)
11. **MANUAL LED:** Illuminates when the shutter is in the MANUAL mode. In MANUAL mode, light output is determined by the Brightness Slide Bar.
12. **Brightness Slide Bar:** Increases light intensity when moved to the right; decreases light intensity when moved to the left.
13. **Voice-Control Port:** Connects to Stryker voice-control systems.
14. **Ethernet Port:** Provides an ethernet connection.
15. **Video-In Port:** Provides a BNC connection for camera video input.
16. **AC Inlet:** Connects to the provided power cord for AC power supply.
17. **Fuse Holder:** Contains two 5A fuses.
18. **Equipotentiality Plug:** Provides a grounding post for common grounding with other equipment.

## Setting Up the X7000

**Note** Your local Stryker Endoscopy sales representative will perform at least one inservice at your convenience to help set up the equipment and instruct you and your staff on its operation and maintenance. Please contact your local Stryker Endoscopy sales representative to schedule an inservice after the equipment has arrived.

**Note** When selecting a setup location for the X7000, consult the “Electromagnetic Compatibility” section included in this manual to determine the best location.

To set up the X7000, make the following connections:

- Connect the AC power cable
- Connect the camera and monitor
- Connect the light cable

### Connecting the AC Power Cable

1. Plug in the AC power cord to the AC Inlet on the rear console panel.
2. Plug in the other end of the AC cord to a hospital-grade outlet.

## Connecting the Camera and Monitor

Figure 2 below illustrates a sample system in which a Stryker medical camera, monitor, and X7000 are connected. The letters in the figure correspond to the letters in the instructions.

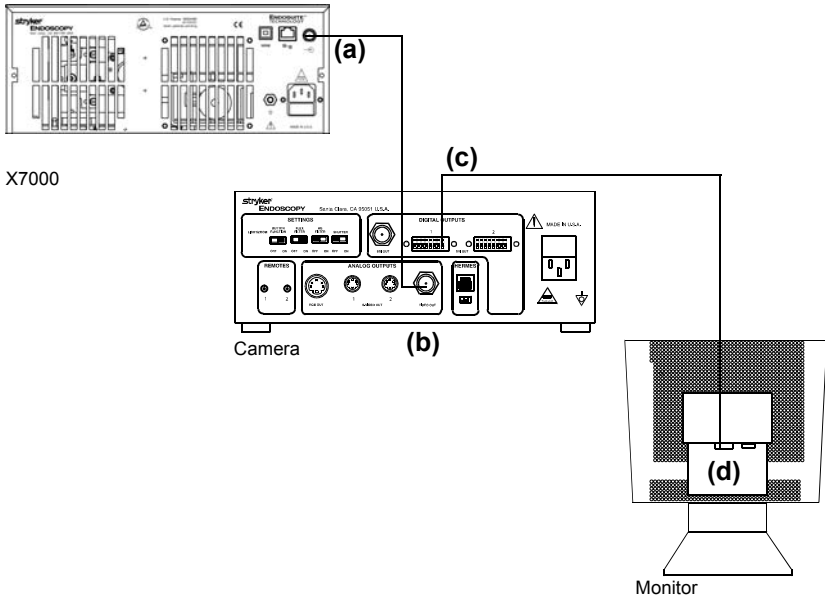


Figure 2: A sample system including a medical camera, monitor, and the X7000

1. Connect a BNC cable to the VIDEO IN port (a) on the rear panel of the X7000.
2. Connect the other end of the BNC cable to the VIDEO OUT port (b) on the rear panel of the camera.
3. Connect a DVI cable to the DVI Out 1 (c) on the rear panel of the camera.
4. Connect the other end of the DVI cable to the DVI input (d) on the rear of the flat-panel monitor.

**Note** BNC cables have push-and-turn connectors. DVI cables have push-only 29-pin connectors.

**Warning** *When the X7000 is interconnected with other medical electrical equipment, leakage currents may be additive. To minimize total patient leakage current, any Type CF applied part should be used together with other Type CF applied parts. Ensure all systems are installed according to the requirements of IEC 60601-1-1.*



# Connecting the Light Cable

**Warning** Use only nonconductive fiberoptic cables with the X7000 to maintain electrical isolation.



**Note** The X7000 Xenon Light Source is compatible with all Stryker Light Cables.

1. Lock open the cable clamp by turning the jaw handle clockwise until it stops (see Figure 3).



Figure 3: Locking open the cable clamp

**Warning** Keep fingers away from the cable clamp as the clamp may inadvertently deploy and cause injury.



**Warning** Do not look directly into the cable port. The high-intensity light may cause damage to the eyes.



2. Insert a clean, dry fiberoptic cable into the cable port until the jaw latch releases and the jaw clamps the cable in place (see Figure 4). Pull gently on the fiber optic cable to test that it is securely seated in the cable port.



Figure 4: Inserting the light cable into the cable port

3. Connect an endoscope to the opposite end of the fiber optic cable.
4. To remove the light cable, turn the jaw handle clockwise until it latches fully open.

**Note** The Light Source will default to STANDBY mode when a light cable is inserted.

# Operating the X7000

**Note** Before operating the X7000, follow the steps in the “Setting Up the X7000” section of this manual.

## Powering the System On and Off

**Caution** *Do not power the system on and off in rapid succession. Allow the lamp to run for at least five minutes once it has been powered on. Failure to do so can cause the lamp to rapidly darken and fail.*



### To power on the X7000:

1. Confirm that the X7000 contains a properly installed lamp. If no lamp is installed, follow the instructions in the “Replacing the Lamp” section of this manual before proceeding.
2. Press the power switch on the front panel. The STANDBY LED will illuminate, indicating the unit is in STANDBY mode.

**Note** The lamp will not illuminate unless a light cable is installed in the cable port.

### To power off the X7000:

1. Disconnect the light cable from the X7000 console.
2. Run the fan for at least one minute to cool the unit.
3. Press the power switch on the front panel of the X7000.

**Warning** *To allow for adequate cooling, never block the rear or side fan vents.*



## Selecting the Operation Mode

The X7000 has two operation modes, RUN and STANDBY.

- **RUN mode:** The RUN mode is used during normal operation. It enables light output to be controlled by the brightness controls on the front console panel.
- **STANDBY mode:** The STANDBY mode is used when the X7000 is powered on but not in immediate use. It reduces the light output to a minimum and minimizes the heat generated at the tip of an unattended light cable or scope.

To select either the RUN or STANDBY mode, press the Mode Switch. The LED for the selected mode will illuminate.

**Warning** *To prevent burns and possible fire, always put the X7000 in STANDBY mode whenever the endoscope is removed from a non-ESST light cable.*



## Adjusting the Brightness

The X7000 has two shutter settings for adjusting the lamp brightness: AUTO and MANUAL.

- **AUTO shutter:** The AUTO shutter automatically adjusts brightness to compensate for variable light levels in the surgical site. Receiving input from the medical camera, the AUTO shutter adjusts light levels around the brightness selected on the Brightness LCD. (For optimal AUTO-shutter performance, use the Brightness Slide Bar to set the brightness between 30 and 70%.)
- **MANUAL shutter:** The MANUAL shutter allows the operator to adjust brightness manually with the Brightness Slide Bar. Move the Brightness Slide Bar to the right to increase brightness and to the left to decrease brightness. The selection will appear on the Brightness LCD as a percentage between 0 and 100.

To select either the AUTO or MANUAL shutter, press the Shutter Switch. The LED for the selected shutter will illuminate.

**Note** For the AUTO shutter to work properly, a medical video camera must be connected to the Video-In Port on the rear X7000 console panel.

**Warning** *The higher the brightness, the more heat energy that will be generated at the endoscope tip. Always adjust the light source to the minimum brightness necessary to illuminate the surgical site.*



**Warning** *The surface temperature near the scope adapter and at the tip of the scope may exceed 41° C if the unit is operated at maximum brightness for extended periods of time. The heated scope and adapter may cause burns.*



## Receiving Feedback from the X7000

The X7000 provides feedback through the Brightness LCD and the Bulb-Hours LCD.

- **The Brightness LCD** shows the intensity level of the light as a percentage between 0 and 100. For example, if the LCD shows “75,” the light output to the fiberoptic cable is running at 75 percent of capacity.
- **The Bulb-Hours LCD** shows how many total hours the lamp has operated. For example, if the LCD shows “250,” the lamp has operated 250 total hours. The Hour-Meter LCD also displays warning and error codes. The table below lists and defines the warning and error codes displayed.


Code	Definition	Recommended Action
E-1	All conditions are met for the lamp to illuminate, yet it remains off.	Return the X7000 for repair.
E-2	All conditions are not met for the lamp to illuminate, yet it remains on.	Return the X7000 for repair.
E-3	The lamp-identification system is not working properly.	Replace the lamp with a Stryker lamp.  If the condition persists, return the X7000 for repair.



Code	Definition	Recommended Action
E-5	The lamp is kept off because the ballast fan-sensing circuit has detected an error.	Return the X7000 for repair.
E-6	The lamp is kept off because the bulb fan-sensing circuit has detected an error.	Return the X7000 for repair.
BULB	The X7000 does not detect a lamp.	Install a Stryker lamp.  If the condition persists, return the X7000 for repair.
OVER	The lamp has significantly exceeded its recommended lifetime.	Install a new Stryker lamp.
WARM	The X7000 is looking for an ethernet connection to check for software upgrades.	See the “Using the Ethernet Interface” section of this manual.

## Safety Shutoff

The X-7000 Xenon Light Source is equipped with a Safety Shutoff feature which will temporarily turn off the bulb in the event of excessive heat.

**Warning**  **WARNING** *Once the Light Source cools down (after 7-10 minutes), power will resume to the bulb and the unit will restart in STANDBY mode. To prevent accidental burns, ensure that the following precaution is taken: Always securely place the scopes and/or fiber-optic cables to assure safe resumption of light output.*

**Note** If the X-7000 Xenon Light Source experiences a temporary shutdown, it is recommended that the device is submitted for service.

**Caution** *Do not abruptly interrupt power to the unit. This will turn off the fan and may cause severe damage to the internal cooling system.*



## Checking the ESST Feature

The X7000 is equipped with Electronic Scope Sensing Technology (ESST), a special safety feature that helps prevent accidental burns caused by an unattended light cable. When operated with an ESST light cable, the X7000 senses when the scope and light cable are separated and places the light source in STANDBY mode. In STANDBY mode, the X7000 will reduce light output to a minimum, preventing the light cable from generating excessive heat.

To verify the ESST feature is active, perform the following test before every surgical procedure:

1. Set up the X7000 system with an ESST light cable and scope, and then power on the system.
2. Place the X7000 in RUN mode.
3. Remove the light cable from the ESST scope adapter.

The X7000 should return to STANDBY mode, indicating that the ESST feature is functioning properly.

**Warning** *If the unit fails to return to STANDBY mode, there may be a fault with the ESST feature. In this case, do not assume ESST safety protection.*



**Warning** *Even with ESST protection, never place the tip of the light cable directly on drapes or on the patient as burns or fire may result.*



## Using the X7000 with a Voice-Controlled System Interface

The X7000 can be used in conjunction with Stryker voice-control systems (Sidne™). For more information about using the X7000 with Stryker voice-control systems, refer to the *Sidne™ Operating and Maintenance Manual* (p/n 1000-400-653).

## Using the Ethernet Interface

The ethernet connection on the rear panel of the X7000 enables remote connection to the Stryker Endoscopy Software Management Site. Connecting to this site enables remote diagnostics and software updates.

**Note** This system feature is not necessary for regular light-source operation.

1. Prior to powering on the X7000, connect one end of a CAT5 ethernet cable to the Ethernet Port on the rear console panel, and the other end to the existing hospital network.

**Note** The hospital network must have a Dynamic Host Configuration Protocol (DHCP) server that can assign an IP address to the unit.

2. Power on the X7000. The unit will search for, download, and install any necessary software updates. It may also transmit diagnostic information. This period will typically take less than one minute. (The Brightness LCD will display the word "WARM" during this period.) Press the MODE Switch to reset the LCD.

**Note** The Ethernet Interface checks for updates each time the X7000 powers on.

# Cleaning and Maintenance

## Cleaning the X7000

**Warning** *Unplug the X7000 before cleaning the unit.*



1. Clean the external surfaces of the X7000 using a cloth or sponge dampened with a mild detergent or disinfectant.
2. Clean and maintain the light cable according to the manufacturer's instructions.

**Caution** *Do not use any abrasive cleaners. Do not allow any liquid to drip into the unit.*



**Caution** *Do not sterilize or immerse the X7000.*



## Caring for the Lamp Module

The X7000 uses a Xenon Cermax lamp, which has a guaranteed life of 500 hours when used properly. Always follow these guidelines to ensure maximum lamp life:

1. Clean the lamp face with alcohol and a cotton swab if it is touched. Dirt or oil on the lamp face will cause the lamp to heat unevenly and fail.
2. Do not power on and off the lamp in rapid succession. Allow the lamp to run for at least five minutes once it has been powered on. Failure to do so can cause the lamp to rapidly darken and fail.
3. The X7000 has been designed to start the lamp under most conditions, even when it is hot. However, if the lamp does not start within 10 seconds after the X7000 has powered on and completed its Ethernet activity (see the "Using the Ethernet Interface" section of this manual), turn the unit off and wait at least five minutes for the lamp to cool before restarting. Further attempts to start the lamp can damage the lamp and possibly the internal circuitry.

## Replacing the Lamp Module

Replace the lamp module when the Bulb-Hours LCD indicates 500 hours or when the lamp no longer sufficiently illuminates the surgical site. If possible, replace the lamp module between surgical procedures.

**Warning** *During operation, the lamp and the housing around the lamp may be hot. Wait at least three minutes for the lamp to cool before handling it.*



To replace the lamp module,

1. Power down the X7000.
2. Open the lamp door (see Figure 5).



Figure 5: Opening the lamp door

**Note** If the unit is on, opening the lamp door will turn off the bulb.

3. Grasp the lamp module by the handle and remove the lamp. (See Figure 6.)

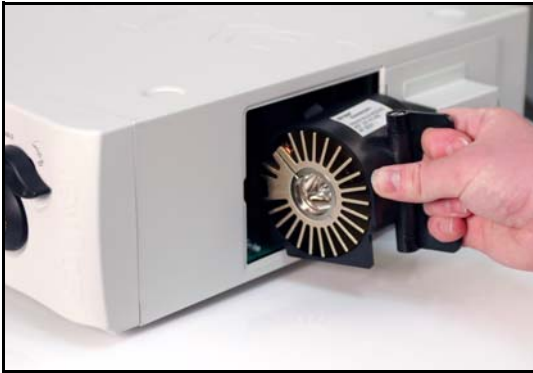


Figure 6: Removing the lamp module

**Warning** *Do not reach inside the lamp door for any reason other than replacing the lamp module. Touching parts other than the lamp module may cause burns or product damage.*



**Warning** *Do not touch any part of the lamp module except for the handle. The module may be very hot and cause burns.*



4. Slide in the new lamp module (Stryker part number 220-191-000) until it is fully seated on the mating connectors.
5. Close and latch the lamp door.
6. Power on the unit.
7. Power on the bulb (if not already on) and verify that the hour meter reads "0."

**Note** Using a non-Stryker replacement bulb module will result in no light output.

**Note** Do not operate the X7000 with a burned out lamp or with no lamp installed.

## Replacing the Fuses

1. Unplug the light source from the AC outlet and remove the power cord from the rear of the unit.
2. Unlatch the fuse holder (13) and remove the fuse(s).
3. Replace the fuse(s) with the same value and rating.

**Warning** *To avoid the risk of fire, use only 5.0A 250V fuses.*



4. Reinstall the fuse holder.

## Disposing of the X7000

The device must be disposed of according to local laws and hospital practices. The device does not contain any hazardous materials.

# Troubleshooting

<b>Problem</b>	<b>Possible Solution</b>
<b>No light output</b>	<ul style="list-style-type: none"><li>• Ensure the AC power cord is properly connected to a hospital-grade power outlet and the inlet on the rear console panel.</li><li>• Ensure the power switch on the front panel is powered on. (It will illuminate when powered on.)</li><li>• Ensure all fuses are operating. See the “Replacing the Fuses” section of this manual for further instructions.</li><li>• Ensure the lamp is properly seated in the lamp housing.</li><li>• Ensure the lamp is in operating condition. Replace the lamp if necessary.</li><li>• Ensure the light cable is correctly engaged with the cable port. As a safety feature, the X7000 will provide no light output unless a fiberoptic light cable is properly seated in the cable port.</li><li>• Ensure the bulb access door is completely shut.</li><li>• Check for error codes E-1, E-3, E-5, E-6, or BULB. See the “Receiving Feedback from the X7000” section of this manual for details.</li><li>• Check that vents are not obstructed.</li><li>• The Safety Shutoff may have been activated. If power to the unit does not resume within 10 minutes, please submit the X7000 for service.</li></ul>



**Too much or too little light output**

- Ensure the light cable is correctly engaged with the cable port.
- Ensure the lamp has adequate lamp life remaining. The display on the Bulb-Hours LCD should be less than 500 total hours.
- Ensure the X7000 is in RUN mode. (The RUN LED should be illuminated.) If necessary, press the Mode button to switch from STANDBY to RUN. If the unit remains in STANDBY,
  1. Ensure the light cable is correctly engaged with the cable port.
  2. If an ESST cable is connected to the X7000, ensure the cable is attached to the scope using an ESST scope adapter.
- If the Shutter is in AUTO mode, use the Brightness Slide Bar to adjust the brightness. Ensure the camera signal is connected properly to the VIDEO-IN connector on the rear console panel. For details, see the “Adjusting the Brightness” section in this manual.
- If the Shutter is in MANUAL mode, use the Brightness Slide Bar to adjust the brightness. For details, see the “Adjusting the Brightness” section in this manual.
- Ensure the fiberoptic cable is transmitting light correctly. Hold the light-source end of the cable up to an overhead room light and look into the scope end of the light cable. If the pattern contains any black spots, the light cable may be worn out and require replacement.
- Ensure the light cable is of an adequate size for the application. The cable diameter may be too small to provide adequate light transmission for the medical video camera in the endoscopic application.

**Excessive glare  
in the video**

- If the Shutter is in AUTO mode, ensure the camera signal is connected properly to the VIDEO-IN connector. Adjust the brightness control setting as necessary.

**Note** Stryker Endoscopy has designed the AUTO-shutter feature of the X7000 for use only with Stryker Endoscopy medical video cameras. The AUTO feature may not provide sufficient light when connected to non-Stryker Endoscopy medical video cameras.

- If the Shutter is in MANUAL mode, ensure the electronic shutter on the camera is operating properly to control video signal brightness. Adjust the brightness control setting as necessary.

## Technical Specifications

### Electrical

Primary:	100 - 240VAC, 50/60 Hz, 450W
Fuses (2):	5.0A 250V

### Dimensions

Height:	5.0" (12.7 cm)
Width:	12.5" (31.8 cm)
Depth:	16.8" (42.7 cm)
Weight:	18.0 lbs. (8.2 kg)

### Fiber Optic Cable Range:

2 mm to 6.5 mm diameter

### Lamp

Type:	300 Watt Xenon Cermax (Elliptical)
Life:	Approximately 500 hours

### Operating Conditions

10 to 40°C  
30% to 75% Relative Humidity

### Transportation & Storage

-20 to 60°C  
10% to 75% Relative Humidity  
700hPa to 1060hPa

### Classifications & Approvals

#### Complies with medical safety standards:

IEC60601-1:1988 + A1:1991 + A2:1995  
CAN/CSA C22.2 No.601.1-M90  
UL2601-1: 2nd Edition

#### Complies with medical EMC standard:

IEC60601-1-2:2001  
Class 1 Equipment  
Type CF applied parts  
Water Ingress Protection, IPX0 - Ordinary Equipment  
Continuous Operation

**Patent Protection**

U.S. #5,850,496 and 6,110,107. Other patents pending.

**Stryker European Representative:**

Stryker France  
ZAC Satolas Green Pusignan  
Av. De Satolas Green  
69881 MEYZIEU Cedex  
France

# Electromagnetic Compatibility

Like other electrical medical equipment, the X7000 requires special precautions to ensure electromagnetic compatibility with other electrical medical devices. To ensure electromagnetic compatibility (EMC), the X7000 must be installed and operated according to the EMC information provided in this manual.

**Note** The X7000 has been designed and tested to comply with IEC 60601-1-2:2001 requirements for EMC with other devices.

**Caution** Portable and mobile RF communications equipment may affect the normal function of the X7000.



**Warning** *Do not use cables or accessories other than those provided with the X7000, as this may result in increased electromagnetic emissions or decreased immunity to such emissions.*



**Warning** *If the X7000 is used adjacent to or stacked with other equipment, observe and verify normal operation of the X7000 in the configuration in which it will be used prior to using it in a surgical procedure. Consult the tables below for guidance in placing the X7000.*



## Guidance and Manufacturer's Declaration: Electromagnetic Emissions

X7000 is intended for use in the electromagnetic environment specified below. The customer or the user of X7000 should ensure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic Environment - guidance
RF emissions CISPR 11	Group 1	X7000 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC61000-3-2	Class A	
Voltage Fluctuations/ flicker emissions IEC61000-3-3	Complies	

**Guidance and Manufacturer's Declaration: Electromagnetic Immunity**


X7000 is intended for use in the electromagnetic environment specified below. The customer or the user of X7000 should ensure that it is used in such an environment.

<b>Immunity Test</b>	<b>IEC 60601 Test Level</b>	<b>Compliance Level</b>	<b>Electromagnetic Environment: Guidance</b>
Electrostatic Discharge (ESD) IEC61000-4-2	±6kV contact ±8kV air	±2,4,6kV contact ±2,4,8kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC61000-4-4	±2kV for power supply lines ±1kV for input/output lines	±2kV line to ground ±1kV line to line	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC61000-4-5	±1kV differential mode ±2kV common mode	±0.5, 1kV differential mode ±0.5, 1, 2kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	<5% Ut (>95% dip in Ut) for 0.5 cycle 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 25 cycles <5% Ut (>95% dip in Ut) for 5 sec.	<5% Ut (>95% dip in Ut) for 0.5 cycle 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 25 cycles <5% Ut (>95% dip in Ut) for 5 sec.	Mains power quality should be that of a typical commercial or hospital environment. If the user of X7000 requires continued operation during power mains interruptions, it is recommended that X7000 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	N/A	Power-frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: Ut is the a.c. mains voltage prior to application of the test level.

**Guidance and Manufacturer's Declaration: Electromagnetic Immunity**

X7000 is intended for use in the electromagnetic environment specified below. The customer or the user of X7000 should ensure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment: Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	<p>Portable and mobile RF communications equipment should be used no closer to any part of the X7000 system, including its cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended Separation Distance</b></p> <p><math>d = 1.17 \sqrt{P}</math></p>
Radiated RF IEC 61000-4-3	3 V/m 80MHz to 2.5 GHz	3 V/m	<p><math>d = 1.17 \sqrt{P}</math> 80 MHz to 800 MHz</p> <p><math>d = 2.33 \sqrt{P}</math> 800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>(a)</sup>, should be less than the compliance level in each frequency range <sup>(b)</sup>.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

(a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast, cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the X7000 system is used exceeds the applicable RF compliance level above, the X7000 system should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the X7000 unit.

(b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and the X7000 System**

The X7000 system is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the X7000 system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the X7000 system as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power (W) of transmitter	Separation distance (m) according to frequency of transmitter		
	150 kHz to 80 MHz $d = 1.17 \sqrt{P}$	80 MHz to 800 MHz $d = 1.17 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.33 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.70	3.70	7.37
100	11.70	11.70	23.30

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.



## Warranty

Stryker Endoscopy warrants the X7000 Xenon Light Source against defects in both materials and workmanship to the registered owner at the time of purchase. All components except the lamp are covered by the warranty for a period of one year from the date of purchase. The lamp module is covered by the warranty for a period of 60 days from the date of purchase.

This warranty does not apply to any unit that has been subject to misuse, abuse, neglect, improper installation or operation, or that has been altered, adjusted, or tampered with by any person other than Stryker Endoscopy authorized service personnel.

The customer is responsible for returning the defective equipment to the factory at his or her own expense. Stryker Endoscopy or its representative will service the unit, repair or replace any defective parts thereof, and return the unit to the customer.

If, upon examination, it is determined that the fault has been caused by misuse or abnormal conditions of operation, the repairs will be billed to the customer in the same manner as out-of-warranty repairs.

Instruments repaired by Stryker Endoscopy will be issued a thirty day warranty against defects in both materials and workmanship, provided the original warranty period has passed. Instruments submitted due to defects in materials and workmanship during the warranty period will be repaired at no charge to the customer.

The warranty as set forth herein is exclusive and in lieu of all other warranties, remedies, obligations, and liabilities of Stryker Endoscopy Inc., expressed or implied, including the implied warranties of merchantability and fitness for use and of consequential damages. These products are sold only for the purpose described herein, and such warranty runs only to the purchaser. In no event shall Stryker Endoscopy be liable for any breach of warranty in any amount exceeding the purchase price of the product.

No agent, employee, or representative of Stryker Endoscopy has the authority to bind the Company to any other warranty, affirmation, or representation concerning this instrument.

This warranty is valid only to the original purchaser of Stryker Endoscopy products obtained directly from Stryker Endoscopy or from a Stryker Endoscopy authorized agent. The warranty cannot be transferred or assigned by the original purchaser.

The X7000 Xenon Light Source warranty is void if any WARNINGS, CAUTIONS, or NOTES are disregarded.

All Stryker products are warranted against defects in materials and workmanship.

© Stryker and Stryker Endoscopy are registered trademarks of Stryker Corporation.

## Service and Claims

This equipment is carefully packaged to prevent damage during transit and is shipped freight outbound (F.O.B.) from San Jose, CA. Therefore, Stryker's responsibility for ensuring damage-free delivery ends upon remission of the equipment to the carrier. Examine the shipment promptly upon receipt.

If the product packaging appears damaged at the time of delivery, immediately check the device for damage. If there is a malfunction, return the device to the distributor.

### **To return equipment damaged during shipment:**

1. File a claim with both the carrier and the distributor.
2. Repack the equipment in its original shipping container and ship it prepaid and insured to:

Stryker Endoscopy Customer Service Department  
5900 Optical Court  
San Jose, CA 95138

### **If service is needed either during or after the warranty period:**

1. Contact Stryker Endoscopy at 1-800-624-4422 or phone your local Stryker Endoscopy sales representative.
2. Package all the components carefully in the original shipping container if possible.
3. Ship the light source, prepaid and insured to:

Stryker Endoscopy Customer Service  
Attention: Repair Department  
5900 Optical Court  
San Jose, CA 95138

**Note** To maximize the longevity, performance, and safety of this device, package it in the original shipping container when storing or transporting.

**Note** The light source described in this manual is continually reviewed, and improvements may be made without notice. A replacement part may not appear the same as the original, but all parts with the same part number will be completely interchangeable.

## Other Service

For service in the U.S.A., call your Stryker Endoscopy representative or call Stryker Endoscopy Customer Service at 1-800-624-4422. Outside of the U.S.A., please contact your Stryker Endoscopy distributor at one of the following locations:

Stryker Corporation  
2725 Fairfield Road  
Kalamazoo, MI 49002  
USA  
Phone:1-269-385-2600  
Telex:224464 STRYKER KMZ  
Fax:1-269-385-1996

Stryker Canada  
45 Innovation Drive  
Hamilton, Ontario, Canada  
L9H 7L8  
Phone: (905) 690-5700  
(800) 668-8323 (toll free)  
Fax: +1(905) 690-5698

Stryker Deutschland GmbH  
Gewerbeallee 18, D-45478  
Mulheim an der Ruhr  
GERMANY  
Phone:49-208-999-060  
Fax:49-208-999-0666

Stryker Latin America  
15100 N.W. 67th Ave. Suite 210  
Miami, Florida 33014  
USA  
Phone:1-305-821-1888  
Fax:1-305-826-0067

Stryker B.V.  
Marinus van Meelweg 17  
P.O. Box 8747  
5657 En Eindhoven  
THE NETHERLANDS  
Phone:31-40-2922522  
Fax:31-40-2922555

Stryker Osteonics, SA  
5, Chemin des Aulx 5  
1228 Plan-les-Ouates  
Case Postale 725  
1212 Grand-Lancy 1  
Geneve, SWITZERLAND  
Phone:41-22-884-0111  
Fax:41-22-884-0199

Stryker European Rep - RA/QA  
Manager  
ZAC Satolas Green Pusignan  
Av. De Satolas Green  
69881 MEYZIEU Cedex, France  
Phone:33-1-48175000  
Fax:33-1-48632175

Stryker India Private Limited  
First Floor  
C-5, SDA Commercial Complex  
New Delhi 110 017  
INDIA  
Phone:91-11-686-6740  
Fax:91-11-696-6020

Stryker Australia  
No. 50 Broughton Road  
Artarmon, NSW 2064  
AUSTRALIA  
Phone:61-2-9415-5100  
Fax:61-29-4294127

Stryker Singapore PTE/LTD  
70 Bendemeer Road  
#03-32 Hiap Huat House  
SINGAPORE 339940  
Phone:65-293-0119  
Fax:65-293-7028

Stryker Pacific Ltd.  
Suite 2501, Citibank Tower  
Citibank Plaza  
3 Garden Road, Central  
HONG KONG  
Phone:61-2-9415-5100  
Fax:61-29-4294127

Stryker Mexico, S.A. de C.V.  
Calle Sacramento 410  
Col. Insurgentes San Borja  
C.P. 03100  
Mexico, D.F.  
MEXICO  
Phone:525-488-0890  
Fax:525-488-0891

Stryker Finland  
PL 80 (Makelankatuz)  
FIN 00501 Helsinki  
FINLAND  
Phone:358 (0) 9 7744 680  
Fax:358 (0) 9 7744 6820

Stryker Korea  
11F Dong Sung Bldg.  
154-24 Samsung-dong  
Kangnam-ku  
Seoul, KOREA 135-090  
Phone:82-2-34517572  
Fax:82-2-552-4156

Stryker China Limited  
Room 903-905, Office Tower 2  
Beijing Sun Dong An Plaza  
138 Wang Fu Jing Da Jie  
Beijing 100006, P.R. China  
Phone:86-10-65136183  
Fax:86-10-83913571

Stryker Japan  
Dai Tokyo Kasai Shinjuku Bldg.  
3-25-3, Yoyogi  
Shibuya-ku, Tokyo 151-0053  
Phone:813-535-29106  
Fax:813-535-21789

Stryker Europe Headquarters  
Cite-Centre, Grand Rue 92  
CH-1820 Montreux  
SWITZERLAND  
Phone:41-21-966-1201  
Fax:41-21-966-1200

Stryker Taiwan  
5F-1,23 Pa Te Road  
Section 1, Taipei, TAIWAN, R.O.C.  
Phone:886-2-2322-2895  
Fax:886-2-2357-8543

Stryker U.K. Ltd.  
Hambridge Road  
Newbury  
Berkshire RG14 5 EP  
United Kingdom  
Phone:44-1635-262400  
Fax:44-1635-262464

Stryker Middle East / Africa  
Via Della Posta  
6934 Bioggio  
Switzerland  
Phone:(4021) 212-1122  
Fax:(4021) 212-1133

NV Stryker SA (Belgium)  
Ikaros Business Park Fase III  
Ikaroslaan 12  
1930 Zaventem  
Brussels, BELGIUM  
Phone:32-2-717-92-10  
Fax:32-2-717-92-49

Stryker Chile  
Avenida Nueva Tajamar 481  
Oficina 805 Piso 8 Torre Norte  
Santiago, CHILE  
Phone:562-244-3600  
Fax:562-244-3696

Stryker Spain  
Manuel Tovar 35  
28034 Madrid  
SPAIN  
Phone:34-91-7283500  
Fax:34-91-3580748

Stryker AB Scandinavia  
Krossverkgatan 3  
S-216 10 Malmö  
SWEDEN  
Phone:46 40-69-18-100  
Fax:46 40-69-18-190

Stryker AB Denmark  
Sankt Annae Plads 9  
1021 Copenhagen, Denmark  
Phone:45 33 9360 99  
Fax:45 33 9320 69

MANUFACTURER  
Stryker Endoscopy Inc.  
5900 Optical Court  
San Jose, CA 95138  
USA  
Phone:408-754-2000  
Fax:408-754-2505



**stryker<sup>®</sup>**  
**ENDOSCOPY**

---

5900 Optical Court  
San Jose, CA 95138  
1-800-624-4422  
1-800-729-2917

1000-400-651 Rev. C